
# Getinge Assured Plasma Indicator

## Directions For Use

GETINGE  
GETINGE GROUP

### INTERPRETATION IMAGES

1

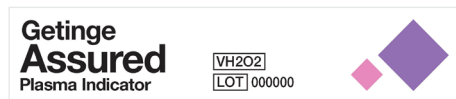2

Any pink indicates exposure to a hydrogen peroxide process

### ENGLISH

#### INDICATIONS FOR USE

The Getinge Assured Plasma Indicator gives visible confirmation that goods have been exposed to hydrogen peroxide.

#### DIRECTIONS FOR USE

1. Place a Plasma Indicator in each pack, peel pouch or tray to be processed in a plasma sterilizer (H<sub>2</sub>O<sub>2</sub>).
2. Sterilize goods as recommended by manufacturer.
3. When unpacking sterilized goods, verify that the indicator diamond symbol has turned from blue to a shade of pink. If the diamond symbol has not sufficiently changed color then resterilization is necessary.

#### COLOR CHANGE

During the process, the indicator diamond symbol changes from blue to a shade of pink.

- 1 Unexposed or fail
- 2 Exposed

#### SAFETY PRECAUTIONS

If there is any doubt about sterility of item, it must be considered not sterile. Do not use any indicator where the diamond symbol is not blue in color prior to exposure to hydrogen peroxide.

#### STORAGE

The Getinge Assured Plasma Indicator should be stored at normal room temperature 10°-38°C and 10-60% Relative Humidity. Do not store near strong oxidizing agents such as chlorine bleach, hydrogen peroxide or peracetic acid. Protect from moisture and excess humidity.

#### EXPIRY DATE

The expiry date is listed on the product packaging. Do not use after the expiration date listed on the product packaging.

#### INTERFERING SUBSTANCES OR CONDITIONS

There are no known interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

#### RELEASE OF TOXIC SUBSTANCES

The indicator releases no known toxic substances in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

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